

## How To Reset Morita 2D CP

It is possible that the arm of the Morita is stuck. You will need to reset the unit.

Follow these instructions :

1. The unit has to be turned **OFF**.
2. **Manually**, push or pull the head anywhere between **min and max height position** if not already there.
3. Switch **ON** the unit.
4. Press the **hidden button shortly** on the left **1cm** of the LED display (enter service mode, see picture)



Don't keep press continuously but just **click** it.

5. Turn the Morita **OFF** and turn back **ON**.

### Serviceman Mode (CPU-2)

